

Ergonomic T-Piece



F&P NEOPUFF SINGLE-USE ERGONOMIC T-PIECE

The Fisher & Paykel Healthcare Ergonomic T-Piece is a unique infant resuscitation circuit that can be used with the F&P Neopuff™ Infant T-Piece Resuscitator.

The Ergonomic T-Piece has been designed with a refined Positive End Pressure (PEEP) Valve orientation to provide comfort and control when delivering breaths.

The Ergonomic T-Piece features an integrated silicone duckbill port. This allows surfactant delivery via an endotracheal tube while providing PEEP. It can also function as a suctioning port.



ERGONOMIC T-PIECE | FEATURES

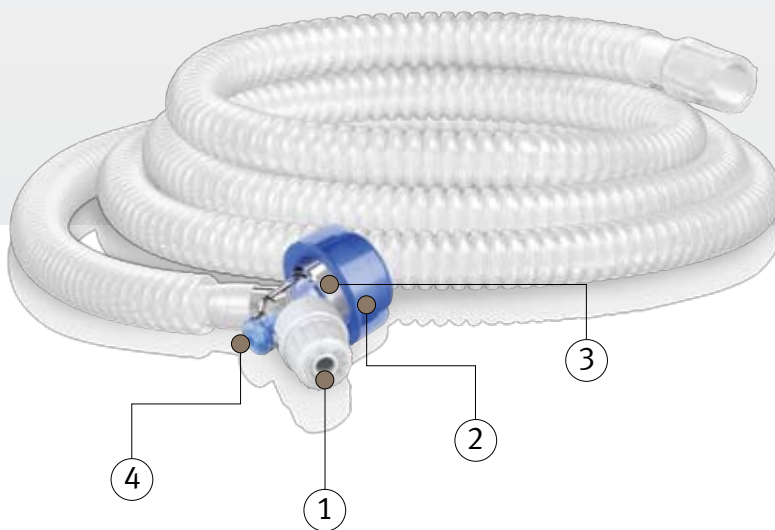

The Ergonomic T-Piece is a unique infant resuscitator circuit for the F&P Neopuff™ Infant T-Piece Resuscitator. It provides a range of features, including a PEEP Valve to accurately set the desired PEEP level, an integrated Duckbill Port and has tubing 1.6 m (63") in length.

PEEP Valve Ergonomics

The Ergonomic T-Piece has been designed with a refined PEEP Valve orientation to provide comfortable and controlled hand positioning when delivering breaths.


Duckbill Port

The integrated Duckbill Port enables surfactant delivery while providing PEEP. Consistent PEEP helps open the infant's lungs to establish Functional Residual Capacity (FRC).


1. Refined PEEP Valve Ergonomics

- Allows for comfortable and controlled hand positioning
- Improved adjustment stability of PEEP Valve to avoid unintentional changes in PEEP




2. Convenient T-Piece Cap

- Set and check pressures reliably
- Can be easily removed for attachment to mask or endotracheal tube
- Provides protective cover for T-Piece on stand-by



3. Versatile T-Piece Connection

- Two options for mask and endotracheal tube connection (15 mm female and 22 mm male)



4. Unique Duckbill Port

- Enables surfactant delivery while providing PEEP
- Allows for the T-Piece to stay in position while suctioning

ERGONOMIC T-PIECE

	RD1300-10	RD1301-10
Box Quantity	10 x Ergonomic T-Piece	10 x Ergonomic T-Piece 10 x Gas Supply Line 2.1 m (82.7")
Tubing Length	1.6 m (63")	
Duckbill Port	5 Fr to 8 Fr (1.7-2.7 mm OD, 0.06"-0.11" OD)	
Material	The product does not contain DEHP or Latex and has an integrated silicone Duckbill Port	
Cleaning	Single patient use – supplied clean, not sterile	
Disposal	Discard according to hospital protocol	

Note: Fisher & Paykel Healthcare also offers Humidified and Classic T-Piece Circuit options. Please refer to the Infant Resuscitation Product Catalog.

www.fphcare.com

Manufacturer ■ Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com **Australia** Tel: +61 3 9879 5022 Fax: +61 3 9879 5232 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 40 216 3555 Fax: +31 40 216 3554 **China** Tel: +86 20 3205 3486 Fax: +86 20 3205 2132 **France** Tel: +33 1 6446 5201 Fax: +33 1 6446 5221 **Germany** Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **India** Tel: +91 80 4284 4000 Fax: +91 80 4123 6044 **Irish Republic** Tel: 1800 409 011 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 3661 7205 Fax: +81 3 3661 7206 **Northern Ireland** Tel: +0800 132 189 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Sweden** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 **UK** (EU Authorised Representative) Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: +44 1628 626 136 Fax: +44 1628 626 146 **USA** Tel: +1 800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001